

MADE IN THE UK



Medipal®

Infection prevention. Everywhere.

Alcohol Wipes

- ✓ MRSA
- ✓ E.coli
- ✓ TB
- ✓ HIV
- ✓ Hepatitis

For disinfecting hard surfaces and non invasive medical devices



With **TRITEX®** TECHNOLOGY



Part of the Medipal healthcare cleaning and disinfection system.

Medipal Alcohol Wipes are made of Tritex non woven and all contain 70% w/w isopropyl alcohol. Ideal for rapid disinfection of non porous hard surfaces and non alcohol sensitive, non invasive medical devices.

Effective against 99.999% of bacteria according to European standard tests; BS EN 1276, BS EN 13704, BS EN 13727, BS EN 14348, BS EN 14476, BS EN 14561, 3-stage Siani protocol C.diff and MRSA test and BS EN 13697. These tests demonstrate that the Medipal Alcohol Wipe range destroys the following microorganisms in the following times:

Product description

Medipal Alcohol Wipes are made of Tritex non woven and all contain 70% w/w isopropyl alcohol.

Just the facts



The most **rigorously tested, effective and proven efficacious** alcohol wipe product on the market.

Tritex technology – strong and low linting which means they're tougher and don't produce fibres, reducing infection risk.

Smear free, leaving no visible residue on glass and stainless steel.

Effective on their own or with other products from the Medipal healthcare cleaning system.

Saves healthcare organisations money. Manufactured in-house, using state-of-the-art machinery and lean techniques, our products cost less to produce and we pass those savings on to our customers.



Controlled dosage ensures disinfection every time.



Canister lids have a tiny tear cross hole – keeping the remaining wipes **moist and preventing them from drying out.**

Proven bactericidal, virucidal, fungicidal and tuberculoidal and effective against common pathogens. See reference table opposite.

Effective and safe for use on non invasive medical devices. These items are rarely cleaned and are a common source of hospital acquired infections. Some items are sensitive to alcohol though so please check the manufacturers' cleaning Instructions prior to use.

Part of the Medipal healthcare cleaning and disinfection system, the most complete range of products on the market.

Widest range available with products to suit every need, in a wide variety of innovative user friendly packaging.

Easy refill packs save time and control your costs.

Microorganism	Testing standard	Contact time	Pass
Bactericidal			
Staphylococcus aureus	EN1276	5 min	✓
Pseudomonas aeruginosa	EN1276	5 min	✓
Enterococcus hirae	EN1276	5 min	✓
Escherichia.coli	EN1276	5 min	✓
Staphylococcus aureus EMRSA	EN1276	5 min	✓
Salmonella choleraesuis	EN1276	5 min	✓
Staphylococcus aureus	EN13727	1 min	✓
Pseudomonas aeruginosa	EN13727	1 min	✓
Enterococcus hirae	EN13727	1 min	✓
MRSA	EN13727	1 min	✓
VRE	EN13727	1 min	✓
MRSA	3 step wipes test	10 sec	✓
Staphylococcus aureus	EN13697	30 sec	✓
Pseudomonas aeruginosa	EN13697	30 sec	✓
Enterococcus hirae	EN13697	30 sec	✓
Escherichia.coli	EN13697	30 sec	✓
MRSA	EN13697	30 sec	✓
VRE	EN13727	30 sec	✓
Acinetobacter baumannii	EN13697	30 sec	✓
Staphylococcus aureus	EN14561	30 sec	✓
Pseudomonas aeruginosa	EN14561	30 sec	✓
Enterococcus hirae	EN14561	30 sec	✓
Escherichia.coli	EN14561	30 sec	✓
MRSA	EN14561	30 sec	✓
VRE	EN14561	30 sec	✓
Acinetobacter baumannii	EN14561	30 sec	✓
Tuberculocidal			
Mycobacterium terrae	EN14563	5 min	✓
Virucidal			
Hepatitis C	Blu Test testing protocol	5 min	✓
HIV	Blu Test testing protocol	5 min	✓
Feline Coronavirus SARS	Blu Test testing protocol	30 min	✓
Yeasticidal and Fungicidal			
Aspergillus niger	EN1650	15 min	✓
Aspergillus niger	EN13624	30 min	✓
Candida albicans	EN1650	15 min	✓
Candida albicans	EN13624	30 min	✓

For disinfecting hard surfaces
and non invasive medical devices.

With
TRITEX[®]
TECHNOLOGY

Tritex is an innovative non woven technology which provides superior performance over other healthcare wipes on the market. Tritex is a three hydrophilic layer structure with the ability to hold in excess of five times its weight of solution. The layers are bonded together in such a way giving it the following characteristics:

- Lowest linting.
- Incredibly strong.
- Highly absorbent.

 **Where to use**

Non Tritex wipe



Easy to tear high linting

Tritex wipe



Hard to tear very low linting

How to clean effectively

Medipal Alcohol Wipes are ideal for disinfecting hard surfaces and non invasive medical devices.

There have never been examples of damage to equipment through use of these products but we do recommend that you refer to the manufacturers' specific instructions before use.

Step 1



Carry out your normal risk assessment, refer to manufacturers' cleaning instructions for the piece of equipment and put on any necessary personal protective equipment.

Step 2



Select which wipe format you want to use and remove the Medipal Alcohol Wipe.

Step 3




Clean your item by going back and forth, downwards in an S shape, being careful not to go over the same area twice. Move from clean to dirty.

Step 4




Change to a new wipe if your wipe becomes soiled or dry.

Step 5

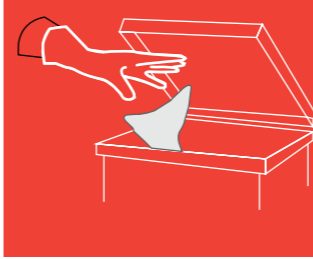


Discard used wipes in the appropriate waste bin.

Step 6

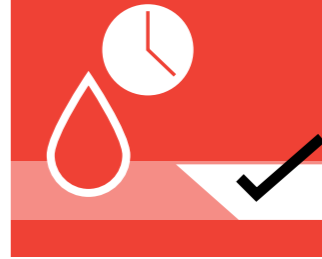

Allow the surface of your item to dry out before use.

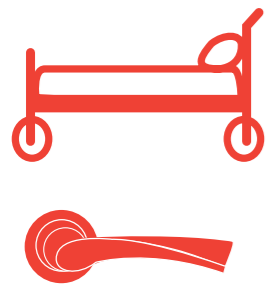

**Non invasive
medical
devices.²**

Surfaces e.g.
countertops, desks,
door handles,
ledges, table tops,
trolleys, cabinets,
drugs trolleys,
nurses' station
and wheelchairs.²



Cleans up small
blood spills.²

Hard non
porous surfaces
and plastics.¹



Equipment e.g.
commodes, keyboards
and accessories,
telephones, walking
frames, bed frames
and chairs.





Alcohol Wipes canister with Tritex

Code	No	Size (mm)
W60510MP	150	130 x 180
W600110MP	225	195 x 210



Alcohol Wipes canister refill with Tritex

Code	No	Size (mm)
Q600110MP	225	195 x 210



Wall mounted brackets

Code	
X49110	Small
X69110	For refills



Alcohol Wipes flow wrap


Code	No	Size (mm)
S605110MP	25	200 x 200
S610110MP	50	200 x 200
S600110MP	100	200 x 200



Alcohol Wipes sachets

Code	No	Size (mm)
SSD600110MP	200	190 x 125

The Medipal Alcohol Wipes form part of a complete system of healthcare cleaning and disinfection products which are designed to complement each other, optimising healthcare cleaning when used together. The following product ranges are available through your usual Medipal distribution partners.

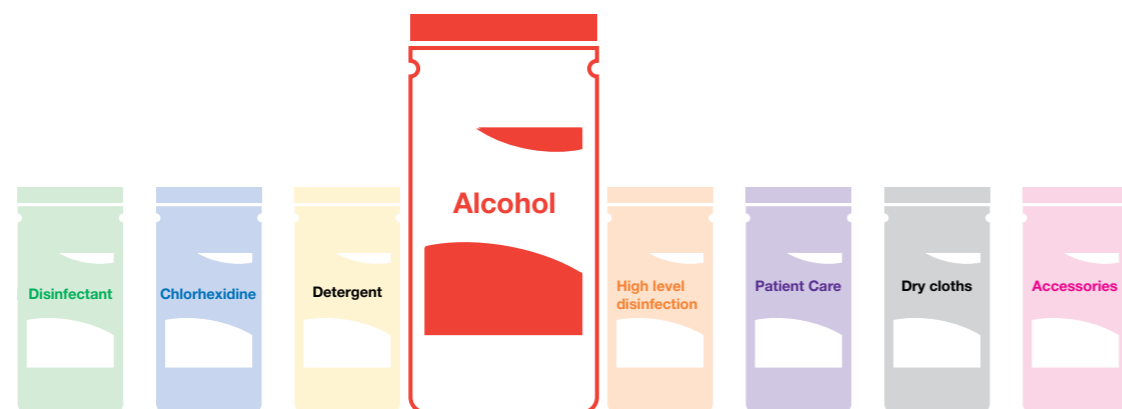

Medipal product	Patient care	Cleaning	Disinfection	High level disinfection
Alcohol	✓	✓	✓	
Chlorhexidine	✓	✓	✓	✓
Detergent		✓		
Disinfection		✓	✓	
Dry cloths		✓		
High level disinfection		✓	✓	✓
Patient Care	✓			
Accessories	✓	✓	✓	✓

Medipal quality

Pal International is a UK manufacturer that has been providing global hygiene solutions for more than 40 years. Our focus on quality and innovation means that the Pal brand is recognised and trusted worldwide.

The Medipal Alcohol Wipe Range has been developed and rigorously tested by our in house microbiologists and chemists before being independently verified and tested to the highest EN standards by the leading laboratories listed below.

The range has been found to be effective against the widest range of pathogens to date.

All Medipal Wipe products are manufactured by Pal International Ltd in the UK. Quality systems are certified to ISO 13485:2003 and our products for medical devices all carry the CE mark.

- BluTest • Cardiff University • Birmingham City Hospital • Abbotts Analytical

References

1. Panousi MN, Williams GJ, Girdlestone S and Maillard JY. (2009) Use of alcoholic wipes during aseptic manufacturing, Letters in Applied Microbiology 48, pp 648-651.
2. NHS Wirral. Cleaning and disinfection policies. Dec 2012: pp 9.